



International Ref. Reagent  
HIV-1 P24 Antigen  
NIBSC code: 90/636  
Instructions for use  
(Version 6.0, Dated 12/12/2023)

This material is not for in vitro diagnostic use

### 1. INTENDED USE

This material is intended for the calibration of secondary reference materials

### 2. CAUTION

**This preparation is not for administration to humans or animals in the human food chain.**

Each individual donation from which the Standard was prepared, and the final freeze-dried Standard, was tested and found negative for HBsAg, anti-HIV, anti-HCV and HCV RNA by PCR. However, as with all materials of biological origin, this preparation should be regarded as potentially hazardous to health. It should be used and discarded according to your own laboratory's safety procedures. Such safety procedures probably will include the wearing of protective gloves and avoiding the generation of aerosols..

As with all materials of biological origin, this preparation should be regarded as potentially hazardous to health. It should be used and discarded according to your own laboratory's safety procedures. Such safety procedures should include the wearing of protective gloves and avoiding the generation of aerosols. Care should be exercised in opening ampoules or vials, to avoid cuts.

### 3. UNITAGE

The assigned potency is 1000 International Units per ampoule.

### 4. CONTENTS

Country of origin of biological material: United Kingdom/United States.

Each ampoule contains the residue after freeze drying of 1ml of a solution containing:

p24 peptide isolated from detergent treated HIV-1

6mg human serum albumin

No bacteriostatic or bacteriocidal agents were added.


### 5. STORAGE

Unopened ampoules should be stored at -20oC.

Please note because of the inherent stability of lyophilized material, NIBSC may ship these materials at ambient temperature.

### 6. DIRECTIONS FOR OPENING

Tap the ampoule gently to collect the material at the bottom (labelled) end. Ensure ampoule is scored all round at the narrow part of the neck, with a diamond or tungsten carbide tipped glass knife file or other suitable implement before attempting to open. Place the ampoule in the ampoule opener, positioning the score at position 'A'; shown in the diagram below. Surround the ampoule with cloth or layers of tissue paper. Grip the ampoule and holder in the hand and squeeze at point 'B'. The ampoule will snap open. Take care to avoid cuts and projectile glass fragments that enter eyes. Take care that no material is lost from the ampoule and that no glass falls into the ampoule.

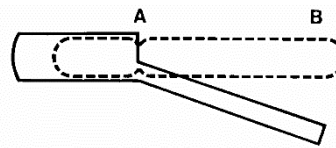

Side view of ampoule opening device containing an ampoule positioned ready to open. 'A' is the score mark and 'B' the point of applied pressure.

### 7. USE OF MATERIAL

**No attempt should be made to weigh out any portion of the material**

For all practical purposes, each ampoule contains the same quantity of the substances listed above. Dissolve the total contents of the ampoule in 1.0 ml of sterile distilled water. The concentration is then 1000 International Units per ml. For further dilution include carrier protein. No attempts should be made to weigh out any portion of the freeze-dried material

### 8. STABILITY

It is the policy of WHO not to assign an expiry date to their international reference materials. They remain valid with the assigned potency and status until withdrawn or amended.

Reference materials are held at NIBSC within assured, temperature-controlled storage facilities. Reference Materials should be stored on receipt as indicated on the label. For information specific to a particular biological standard, contact the Technical Information Officer or, where known, the appropriate NIBSC scientist. In addition, once reconstituted, diluted or aliquoted, users should determine the stability of the material according to their own method of preparation, storage and use.

NIBSC follows the policy of WHO with respect to its reference materials.

Users who have data supporting any deterioration in the characteristics of any reference preparation are encouraged to contact NIBSC.

Reference materials are held at NIBSC within assured, temperature-controlled storage facilities. Reference Materials should be stored on receipt as indicated on the label.

NIBSC follows the policy of WHO with respect to its reference materials.

### 9. REFERENCES

None

### 10. ACKNOWLEDGEMENTS

### 11. FURTHER INFORMATION

Further information can be obtained as follows;

This material: [enquiries@nibsc.org](mailto:enquiries@nibsc.org)

WHO Biological Standards:

<http://www.who.int/biologicals/en/>

JCTLM Higher order reference materials:

<http://www.bipm.org/en/committees/jc/jctlm/>

Derivation of International Units:

[http://www.nibsc.org/standardisation/international\\_standards.aspx](http://www.nibsc.org/standardisation/international_standards.aspx)

Ordering standards from NIBSC:  
<http://www.nibsc.org/products/ordering.aspx>  
 NIBSC Terms & Conditions:  
[http://www.nibsc.org/terms\\_and\\_conditions.aspx](http://www.nibsc.org/terms_and_conditions.aspx)

## 12. CUSTOMER FEEDBACK

Customers are encouraged to provide feedback on the suitability or use of the material provided or other aspects of our service. Please send any comments to [enquiries@nibsc.org](mailto:enquiries@nibsc.org)

## 13. CITATION

In all publications, including data sheets, in which this material is referenced, it is important that the preparation's title, its status, the NIBSC code number, and the name and address of NIBSC are cited and cited correctly.

## 14. MATERIAL SAFETY SHEET (Add or amend as necessary)

Classification in accordance with Directive 2000/54/EC, Regulation (EC) No 1272/2008: Not applicable or not classified

Physical and Chemical properties	
Physical appearance: Freeze-dried	Corrosive: No
Stable: Yes	Oxidising: No
Hygroscopic: No	Irritant: No
Flammable: No	Handling: See caution, Section 2
Other (specify):	Contains p24 antigen isolated from detergent-treated HIV-1
Toxicological properties	
Effects of inhalation:	Unknown - Avoid
Effects of ingestion:	Unknown - Avoid
Effects of skin absorption:	Unknown - Avoid
Suggested First Aid	
Inhalation:	Seek medical advice
Ingestion:	Seek medical advice
Contact with eyes:	Wash with copious amounts of water. Seek medical advice
Contact with skin:	Wash thoroughly with water.
Action on Spillage and Method of Disposal	
Spillage of ampoule contents should be taken up with absorbent material wetted with an appropriate disinfectant. Rinse area with an appropriate disinfectant followed by water. Absorbent materials used to treat spillage should be treated as biological waste.	

## 15. LIABILITY AND LOSS

In the event that this document is translated into another language, the English language version shall prevail in the event of any inconsistencies between the documents.

Unless expressly stated otherwise by NIBSC, NIBSC's Standard Terms and Conditions for the Supply of Materials (available at [http://www.nibsc.org/About\\_Us/Terms\\_and\\_Conditions.aspx](http://www.nibsc.org/About_Us/Terms_and_Conditions.aspx) or

National Institute for Biological Standards and Control,  
 Potters Bar, Hertfordshire, EN6 3QG. T +44 (0)1707 641000, [nibsc.org](http://www.nibsc.org)  
 WHO International Laboratory for Biological Standards,  
 UK Official Medicines Control Laboratory

upon request by the Recipient) ("Conditions") apply to the exclusion of all other terms and are hereby incorporated into this document by reference. The Recipient's attention is drawn in particular to the provisions of clause 11 of the Conditions.

## 16. INFORMATION FOR CUSTOMS USE ONLY

<b>Country of origin for customs purposes*:</b> United Kingdom
* Defined as the country where the goods have been produced and/or sufficiently processed to be classed as originating from the country of supply, for example a change of state such as freeze-drying.
<b>Net weight:</b> 1ml when reconstituted
<b>Toxicity Statement:</b> Unknown
<b>Veterinary certificate or other statement</b> if applicable.
<b>Attached:</b> No